

### Klorsept and Klorkleen validated guidelines

Area of use	Type of uses	Method of use
<b>Food Preparation</b>	Washing minimally processed foods	Leave to soak for 15 minutes in a Klorsept solution.
	Food preparation surfaces	Wipe over surfaces with a Klorsept (disinfection only) or Klorkleen solution (cleaning and disinfection), and leave for 15 minutes before use
<b>General Environment</b>	Intermediate and Low Risk	Wipe over surfaces with a Klorsept or Klorkleen solution, and leave for 15 minutes before use
	Diagnostic rooms, patient areas, hydrotherapy pool surrounds, trolleys, instrument trays, medical equipment handles etc.	Immerse items for 15 minutes
	Bedpans, linen, mops etc.	
<b>High Risk</b>	Operating theatres, post-mortem rooms, burns unit, intensive care, isolation units, clinical and pathology laboratories etc.	Clean surfaces and then wash/wipe surfaces with a Klorsept or Klorkleen solution. Leave for 15 minutes. Metallic surfaces should be rinsed with potable water afterwards
	Laboratory discard jars, pipettes etc.	Submerge discard items for 60 minutes before disposal
<b>HIV, HBV and Similar</b>	Contaminated surface or equipment	Wash or wipe surfaces with a Klorsept or Klorkleen solution and leave for 15 minutes. Metallic surfaces should be rinsed with potable water afterwards.

	Body fluid spillage (not urine)	Shake Klorsept Granules on the spillage to completely cover and absorb the fluid. Wait 2-3 minutes and scoop up absorbed product. Wipe over the surface with a Klorsept or Klorkleen solution of 1000mg/l. Dispose of waste.
		Where granules are not available, the area should first be covered with paper towelling or other absorbent material and a Klorsept solution poured over the absorbent material and left for 10 minutes. The whole spill is then wiped up with fresh absorbent material and placed in a contaminated-waste container. The surface is then wiped with a Klorsept/Klorkleen solution of 1000mg/l

Solution strength
50 mg/l
100mg/l
200mg/l
1000mg/l
2500mg/l
5,000mg/l

Granules

10,000mg/l